


**INSTRUMENT GRADE DISINFECTANT – LOW LEVEL**  
Cleaning and disinfection of surfaces  
**ARTG No. 293206**

as at: 30.10.2017 – AU

**Alcohol-free wet wipes for the cleaning and disinfection of sensitive surfaces of medical devices such as dental units incl. upholstery and suspension tables, X-ray machines, instruments and acrylic glass.**

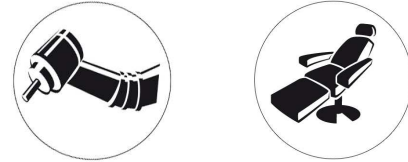

- *tearproof nonwoven wipes of high quality „Cotton Touch Quality“ (50 g/m<sup>2</sup>)*
- *optimal dispensing of active solution*
- *effective cleaning by means of micro pore technology*
- *extra large wipes of 21 x 26 cm, ideal for large surfaces*
- *universal application*
- *without allergenic or sensitizing resp. material damaging active agents as: alcohol, aldehydes, phenols and QAC*



CE 0123

**Composition of soaking liquid:**

Dodecyldipropylentriamine 0.2 - 1 %  
N-C12-14 Alkyltrimethylenediamines  
0.2 – 1 %  
deion. water, fragrances, emulsifying agent

**Contact times**

yeasticial	1 min.
bactericidal + MRSA	5 min.
virucidal, Vaccinia	5 min.

**Efficacy/Certification**

- *bactericidal – e. g. Staphylococcus aureus (incl. MRSA), Pseudomonas aeruginosa, Enterococcus hirae*
- *yeasticial – e. g. Candida albicans*
- *virucidal – e. g. Vaccinia (representing enveloped viruses acc. to EN standards)*  
*with remanence effect/preservation (reduction of microbial recontamination)*

**Clinical data**

Prof. Dr. H.-P. Werner, Schwerin:

- VAH\*, 2014-05
- EN 13624 + EN 13727, 2014-03
- virucidal (carrier test, acc. to DVV\*\* 2012) against enveloped viruses, 2015-01
- Dr. H. Brill, Hamburg, EN 16615, 2016-01

*In special cases clinical data and approvals may be requested from us.*

\*VAH – Association of Applied Hygiene  
\*\*DVV – German Association for the Control of Virus Diseases

**INSTRUMENT GRADE DISINFECTANT – LOW LEVEL**  
Cleaning and disinfection of surfaces  
**ARTG No. 293206**

as at: 30.10.2017 – AU

### **Field of application**

PlastiSept® eco JumboWipes 50, soaked with PlastiSept® eco (ARTG No. I 164389), are particularly material-friendly disinfectant wipes for the special application on surfaces of medical devices sensitive to alcohol, as dental units (e. g. upholstery, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG) and surfaces of acrylic glass (also for solarium beds).

Also usable by persons who react sensitively to alcoholic quick disinfectants.

**Application only by qualified personnel wearing protective clothing.**

### **Shelf life**

2 years

After opening the PlastiSept® eco JumboWipes 50 are usable for at least 28 days – provided that the dispenser lid of the box is always closed tightly

### **Environment**

The active solution in PlastiSept® eco JumboWipes 50 is biodegradable according to OECD guidelines.

### **Material compatibility**

Free of alcohol, aldehydes, QAC and halogen, therefore particularly mild to surfaces.

### **Delivery forms**

REF 4621      Box with 70 wipes  
(50 g/m<sup>2</sup>, 21 x 26 cm),  
soaked with PlastiSept® eco

REF 4622      Cardboard box with 12 refills  
with 70 wipes each  
(50 g/m<sup>2</sup>, 21 x 26 cm),  
soaked with PlastiSept® eco

### **Accessories**

REF 4602      Wall holder for  
Jumbo-Wipes-Box