

Dentists...
isn't it time



veneerTM

dental art made simple



Uveneer™ – Unique template System Produces Glossy, Contoured Direct Veneers

Cosmetic dentistry is now available for more of your patients. Uveneer creates predictable direct aesthetic veneers in a quick, easy, and affordable way. The Uveneer delivers beautiful anatomy, form and function to create natural looking restorations.

Q: How does Uveneer work?

A: A translucent template is pressed over the composite that has been laid on the tooth, the composite is cured through the template. The template is then removed and can be reused as it is fully autoclavable. The template is easy to handle and has a non-stick surface.

Q: When is Uveneer used?

A: For immediate direct composite restorations, mock-ups (to show patients a new cosmetic smile), class 5 restorations and for excellent temporaries during porcelain veneer creation.

Q: Is the Uveneer reusable? Autoclavable?

A: Yes, the templates are autoclaved after each use and can then be reused.

Q: Can Uveneer increase practice profitability?

A: Yes, dentists who use Uveneer report that they are performing many more composite veneer procedures than before because they find the process so easy, quick, predictable and affordable for their patients and with high case acceptance.

Q: What are the main advantages of using Uveneer?

A: The main advantages of Uveneer are that: the dentist can use any preferred composite material; the results are consistent and predictable with regard to final tooth shape, shine and smile design; it is very easy to use and saves valuable time because there is no need to spend time carving or polishing and there is minimal tooth preparation. It is also cost effective for both patients and dentists.

Q: What sizes are the templates?

A: The Uveneer template kit consists of 32 templates: 16 medium templates, 16 large templates. Both medium and large templates provide 2 central incisors, 2 lateral incisors, 2 canines & 2 premolars templates for both the upper and lower arches.



It's time to work faster and smarter!

Uveneer

Typical Uveneer™ Case

Figure 1:

Before. This young woman was embarrassed to show her teeth. Left lateral incisor is an implant crown restoration with poor match to the rest of the dentition.

Figure 2:

Uveneer template is tried on and selected based on correct size, shape and length.

Figure 3:

Cord is packed, and tooth is separated and isolated with matrices. Etched, rinsed, bonding agent applied, dried, and light cured.

Figure 4:

Restorative composite is placed onto tooth & flowable composite is placed onto Uveneer. Uveneer is pressed into place, excess composite is removed with a probe.

Figure 5:

Pressed and cured restoration through Uveneer.

Figure 6:

Removed Uveneer template, retraction cord, and residual excess from periphery.

Figure 7:

Left central veneer is complete and glossy with no polishing necessary.

Figure 8:

Procedure was repeated for right central and right lateral.

Figure 9:

After. A great smile achieved through the new direct composite veneers, using Uveneer system.

Dentists all over the World are raving about Uveneer:



Dr Gary M. Radz

D.D.S, F.A.C.E

“Terrific tool to quickly and easily create beautiful anterior restorations.”



Professor Ian Meyers

Honorary professor, The University of Queensland

“The Uveneers are an excellent aid in restoring anterior teeth and provide an enormous benefit in terms of shaping, angulation and proportioning of the anterior teeth when undertaking labial veneer restorations.”



Dr Martin B. Goldstein

D.M.D, F.A.G.D

“Consider it a Must have!”



Dr Elliott Mechanic

D.D.S

“This product gives me the form and symmetry, you can't loose with it”



Dr John C. Comisi

D.D.S, M.A.G.D.

“The simplicity of Uveneer is absolutely remarkable. Why didn't I think of this?”



Dr Jack Ringer

D.D.S

“It's very quick and efficient, I highly recommend it”

Uveneer.com



info@uveneer.com

Visit our website
to see U-Veneer
in action.



veneerTM
dental art made simple

